


# THIS INSERT REPLACES PAGE 2 OF THE SPL MANUAL

## Features and Benefits

- Automatic Locking Prior To Heel Strike
- Closely Mimics Normal Gait Pattern
- Free Flexion During Swing Phase
- Minimizes Energy Consumption
- Eliminates Circumduction and Vaulting
- Provides Toe Clearance During Swing Phase
- Added Patient Security
- **Increased Range of Motion to 151°**
- **2 Locking Positions at 0° and 15°**
- **Extended Wear Bumper**
- **Teflon Bushings for Sideloading**
- **High Load PEEK Bushing**
- 4 Modes of Control
- No Cables or Special Heel Connections
- Easy Fabrication

## New Features of the SPL 2

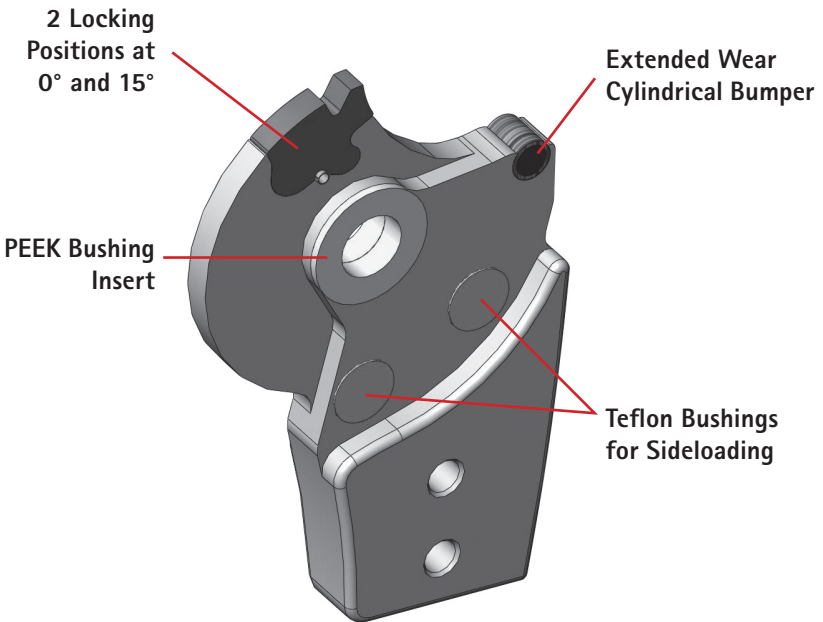

## Warranty

Fillauer orthotic components are warranted for 12 months from the date of shipment from the Fillauer warehouse or from an authorized Fillauer distributor. Items under warranty will be replaced or repaired (at Fillauer's discretion) at no charge. The warranty will be void if the item has been fabricated or installed outside Fillauer's recommendations, if the item has been exposed to a corrosive environment, or if the item has been used in extremely abusive activities that could result in injury.

## Indications:

- Post-Polio
- Spinal Involvement
- Cerebral Vascular Accident
- Peripheral Paresis/Paralysis
- Nerve Inflammations
- Neurological Failures
- Myopathies
- Multiple Sclerosis (or similar diseases)

## Contraindications:

- Knee Flexion Contracture greater than 10°
- Central Paralysis
- Hip Flexion Contracture
- Hip Musculature Involvement
- Poor Balance or Coordination
- Knee Hyperextension greater than 10°

## Locking Modes

Effective immediately all SPL Knee Joints are being executed with the new modified Satellite. The Satellite has the following features:

- Automatic lock mode (Position 1)
- Unlock mode (Position 2)
- Free motion mode (New) (Position 3)
- Lock mode (Position 4)

NOTE: If the mode switch is pressed when in Free Motion Mode, the satellite will drop into the Lock Mode. The Automatic Lock must be re-engaged with the slide lever.



Position 1



Position 2



Position 3



Position 4